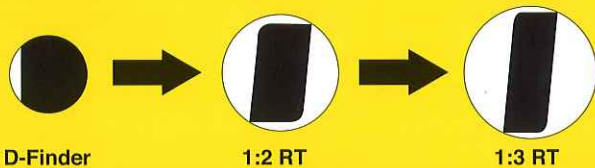
CE

MANI®

FOR ADVANCED TREATMENT

RT FILES SYSTEM (HAND TYPE)

D-Finder

1:2 RT

1:3 RT

Feel the difference

Sharpness & Flexibility

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MANI, INC.

For advanced treatment ——— We serve with micro processing technology

切れ味・フレキシビリティを極めたステンレスファイル Stainless Steel File developed through the investigation of sharpness and flexibility.

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Since the RT Files have cutting edges placed at 85-degree and a revised parallelogram cross section with a large difference between the long side and the short side, they are more flexible and have more cutting ability than conventional files. That's why RT Files make it possible to open up root canal accurately and quickly.

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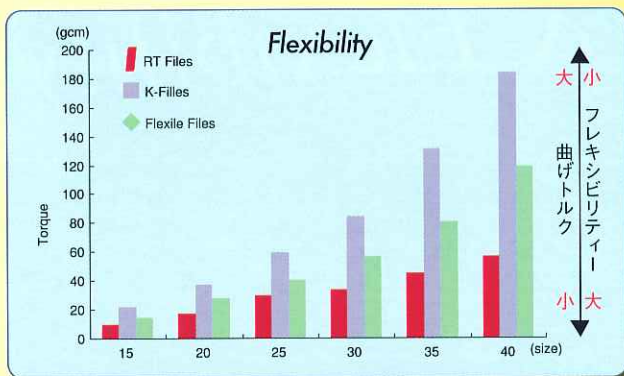
If the land area of a square is assumed to be 1.00, the land area of RT Files is 0.80 (1:2RT) and 0.60 (1:3RT) (Fig. 1). The bending torque is reduced (Table 1) and the proper flexibility is achieved (Fig. 2). That's how they are able to work so well in curved root canals.

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

RT Files have a working angle of 71-degree, which is substantially larger than the 45-degree angle in square cross-section files (Fig. 1). They have more cutting ability than standard 45-degree square files. Therefore, they are able to open up a root canal in an optimum fashion.

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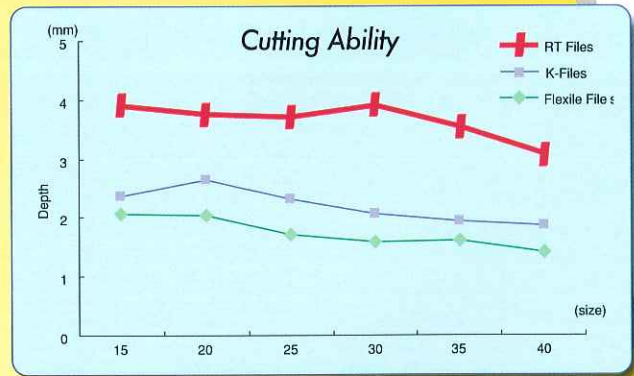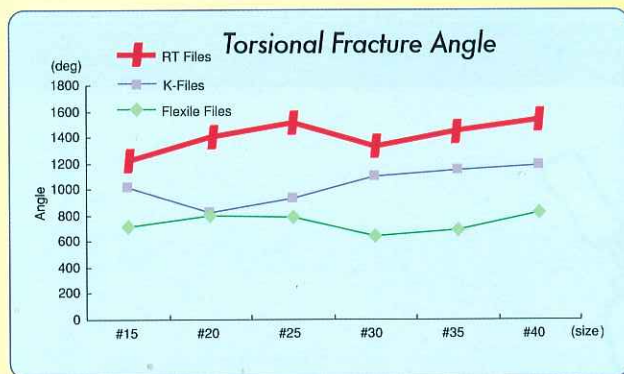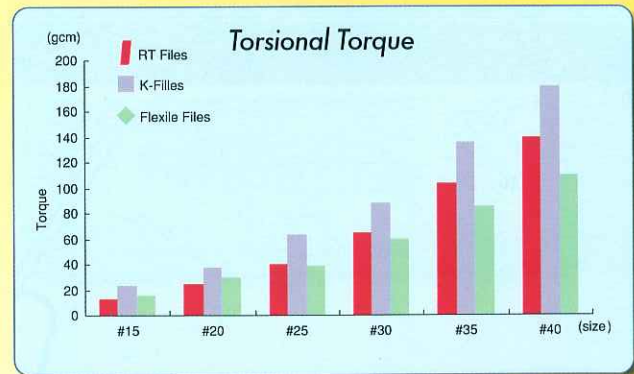
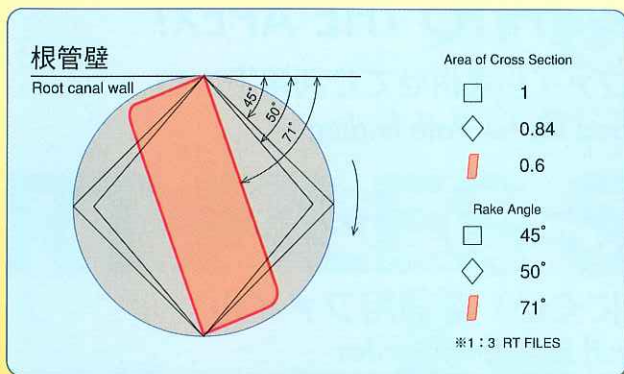
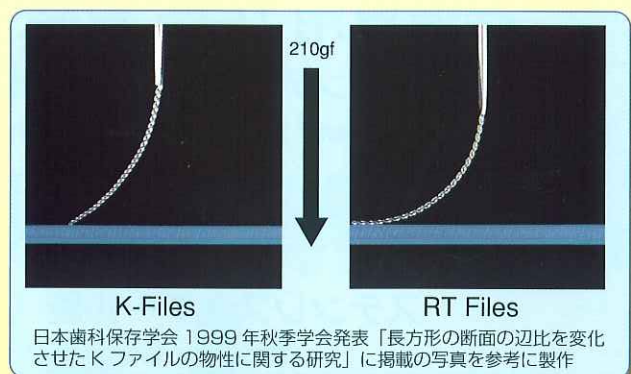
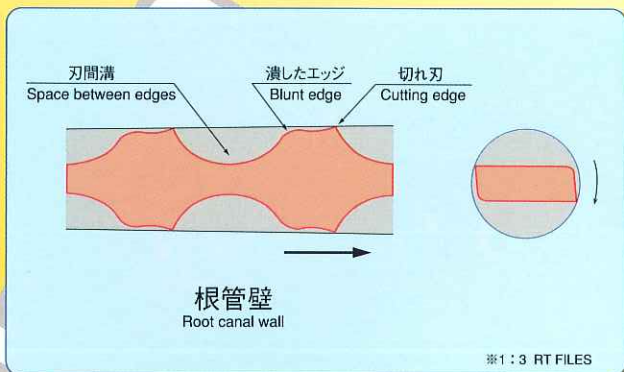
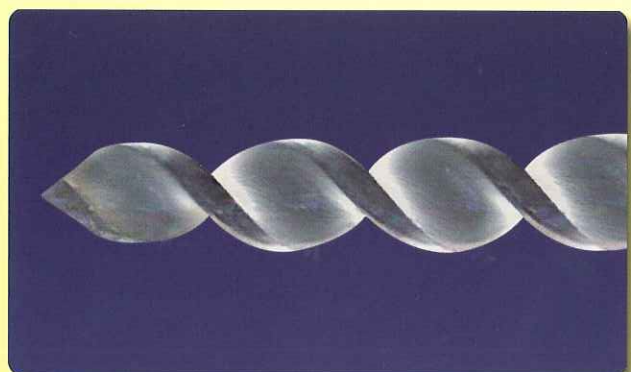
Since the cutting edges of the RT Files are set at large angle of 85-degree, they maintain excellent cutting performance over time. With their deep spaces between the cutting edges and with concaveness of the back edges (Fig. 3), there are few possibilities for pushing the debris toward root apex and the danger of forcing it out of the apical foramen is greatly reduced.

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NiTi alloy files tend to go back to original position due to their shape memory effect, which means that they are in constant danger of cutting away excessive amounts from the outside curve of a root canal. However, since RT Files are made of stainless steel, they can be pre-curved to match with the curvature of a root canal, which lets them follow the natural path accurately.

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The Torsional Fracture Angle Test (Fig. 3) and Torsion Torque Test (Fig. 4) show that the RT Files have more fractural angle than "Other K-Files". Also, the RT Files have higher fractural torque than "Other K-Files". The RT Files give safer advantage in curved root canal.

RT FILES		SIZE	Color	SIZE	Color	SIZE	Color
Length (L)	:21, 25, 28, 31mm	15 (1)		35 (5)		55 (9)	
Packed	:6 pcs. in a package	20 (2)		40 (6)		60 (10)	
Size	:(a) 6 pcs. of one size	25 (3)		45 (7)		70 (11)	
	:(b) Assorted 15(1)~40(6), 45(7)~80(12)	30 (4)		50 (8)		80 (12)	
			(Side Ratio) 1:2		(Side Ratio) 1:3		

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Caution When using RT Files and D Finders please note the following ●This product is designed only for dental use. ●Before using a file, be sure to sterilize it to prevent infection. ●Before using a file, always check it before putting it in the patient's mouth to be sure if it is not deformed, damaged or cracked. ●This product should be treated as medical waste when disposed.